



ELECTRO- MAGNETIC FIELDS TO SCULPT

Unique performance
Customized treatments
Impressive results

CRISTAL®
Fit

FRENCH
TECHNOLOGY

CE
2797

A product from the
CRISTAL Body-Layering® range



1

CRISTAL Pro

MEDICAL CRYOLIPOLYSIS DEVICE
ACTS ON **THE FAT** TO REFINE
CRYSTALLISES AND DESTROYS FAT CELLS.

VOLUME

2

CRISTAL Fit

HIGH INTENSITY ELECTROMAGNETIC MUSCLE
STIMULATION (HI-EMMS) DEVICE
ACTS ON **THE MUSCLE** TO SCULPT
TONES AND SHAPES MUSCLES.

STONE

3

CRISTAL Skin

RADIO-MULTIFREQUENCY SYSTEM
ACTS ON **THE SKIN** TO FIRM UP
TIGHTENS AND REDUCES SAGGY SKIN.

FIRMNESS

CRISTAL®
Body-Layering®

CRISTAL BODY-LAYERING® PROGRAM

A GLOBAL APPROACH TO THE BODY THROUGH THE
COMBINATION OF ADVANCED TECHNOLOGIES

Deleo invents CRISTAL Body-Layering®, a global non-invasive and personalised treatment which acts on the 3 layers of the body: dermis, hypodermis and muscle.

This 3-step program acts on the 3 key parameters for a harmonious body: **VOLUME**, **tone** & **FIRMNESS**.



1

CRISTAL Pro

MEDICAL CRYOLIPOLYSIS DEVICE
ACTS ON **THE FAT** TO REFINE
CRYSTALLISES AND DESTROYS FAT CELLS.

VOLUME

2

CRISTAL Fit

HIGH INTENSITY ELECTROMAGNETIC MUSCLE
STIMULATION (HI-EMMS) DEVICE
ACTS ON **THE MUSCLE** TO SCULPT
TONES AND SHAPES MUSCLES.

TO NE

3

CRISTAL Skin

RADIO-MULTIFREQUENCY SYSTEM
ACTS ON **THE SKIN** TO FIRM UP
TIGHTENS AND REDUCES SAGGY SKIN.

FIRMNESS

CRISTAL®
Body-Layering®

DISCOVER



Second step of the CRISTAL Body-Layering® program,
CRISTAL Fit will act on the tonicity by shaping the muscles.

CRISTAL Fit goes even further in global patient care thanks
to the High Intensity Electromagnetic Muscle Stimulation
technology.

Expand your technical platform with the **CRISTAL Fit** device
and offer your patients results on figure toning.





TECHNOLOGY

A STEP FORWARD IN
FIGURE TONING



EFFICIENCY

- UNIQUE TREATMENT
PARAMETERS
- ERGONOMIC APPLICATORS



PERFORMANCE

A PREMIUM INTERFACE



UNMATCHED SATISFACTION

NEVER-SEEN-BEFORE
RESULTS

BREAKTHROUGH TECHNOLOGY

to tone and shape the body

4/10 PEOPLE
are moderately or not
satisfied with their body.¹

BY 2023
the market will grow by
8.3% per year.³

42% OF PEOPLE
consider the stomach to be
their least favourite body part,
followed by the thighs for

15% OF THEM²



A STEP FORWARD

in figure toning

INNOVATIVE TECHNOLOGY

The **CRISTAL Fit** device is a Body-Layering® system that uses **High Intensity Electromagnetic Muscle Stimulation (HI-EMMS)**.

This technology uses a focused electromagnetic field, the intensity of which is measured in Tesla, based on Faraday's Law of Electromagnetic Induction.

The electromagnetic fields, with a wide and deep spectrum, will allow to reach many types of muscles.

CRISTAL Fit has a medical indication for the rehabilitation of the pelvic floor muscles (MPP) by non-invasive electromagnetic stimulation of the **pelvic floor for the treatment of urinary incontinence (UI)**.

CRISTAL Fit also has the following non-medical indications:

- **CRISTAL Fit** can be used to tone the abdominal muscles, thighs and buttocks.

HOW DOES IT WORK?

The electromagnetic field passes through the body in a **non-invasive and non ablative** manner. It interacts with the motor neurons to trigger **forced and intense muscle contractions** similar to the ones experienced during real

exercise: abdominal «**crunches**» and «**squats**» for the buttocks.

The technology uses a **specific range of frequencies** that do not allow muscle relaxation between two consecutive stimulations, so the muscles remain in a forced state of contraction. Unlike voluntary muscle contractions, forced contractions are independent from brain function and are thus more effective.

As a result, cells are activated to **develop muscle mass** and the figure is toned and takes shape.

As the existing muscle fibers are submitted to intense exercises, a structural muscle modification will be essential to allow to support these exercises repeatedly in the long run. Therefore, the volume of muscle fibers will increase (**hypertrophy phenomenon**) as well as their number or density (**hyperplasia phenomenon**).



CRISTAL FIT TREATMENT PLAN

PATIENT TYPE?

For men and women who are not overweight and who are looking to strengthen their muscles.

DURATION?


30 minutes on average, according to the protocol.

HOW MANY SESSIONS?


6-8 sessions are recommended.

FREQUENCY OF SESSIONS?


Twice a week, depending on how the patient is feeling after each session.

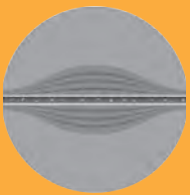


1. The muscle is resting

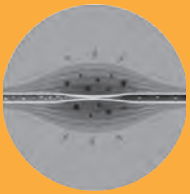

2. Induced
electromagnetic field &
nerve stimulation.

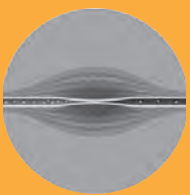



3. Forced and intense
muscle contractions
causing hypertrophy
or hyperplasia
phenomenon.

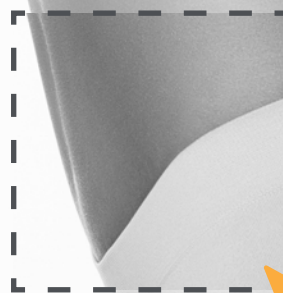

Abdomen

Thighs

Buttocks

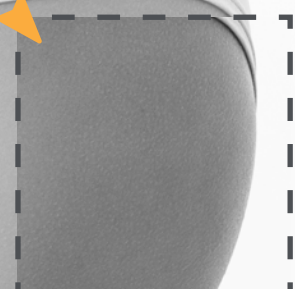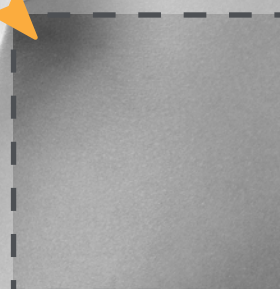

Unique treatment PARAMETERS

INTENSITY

CRISTAL Fit offers the **highest Tesla intensity per applicator** on the market. This will allow you to achieve very intense muscle contractions for more results.


DEPTH

Thanks to its high intensity and its specific pulse frequency, **CRISTAL Fit** can treat **all the areas of the body, in depth**, for homogeneous and lasting results.


PULSE DURATION

With **CRISTAL Fit** you can achieve longer contractions thanks to its **420µs pulse duration** that optimizes treatment efficiency.


CRISTAL Fit is 2 TIMES MORE POWERFUL¹

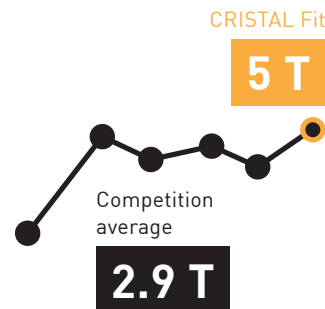

CRISTAL Fit has a LONG PULSE DURATION¹

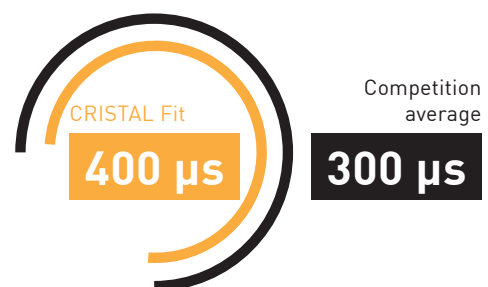

CRISTAL Fit has an OPTIMAL COOLING SYSTEM¹

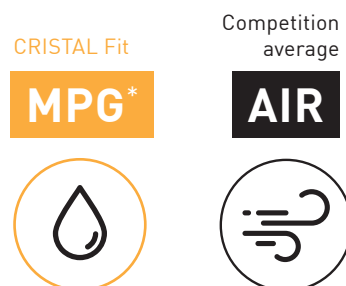

MORE INTENSE
DEEPER
LONGER

PULSE



INCREASED EFFICIENCY



Ergonomic APPLICATORS

IDEALLY DESIGNED APPLICATORS

The **CRISTAL Fit** device features two applicators working simultaneously. Their ergonomic design offers perfect positioning and a comfortable treatment with maximum efficiency. Additionally, they disconnect easily for cleaning.

COMING SOON!




Future patented innovations are on their way... Deleo's engineers are working on future patents to offer you a perfect range of applicators for various indications.



ADVANCED COOLING SYSTEM

CRISTAL Fit is equipped with a cooling system inside its applicators to offer a safe treatment and ensure device durability in the long run. It is thus possible to apply a high power without creating an uncomfortable sensation of heat for the patient.

A premium INTERFACE

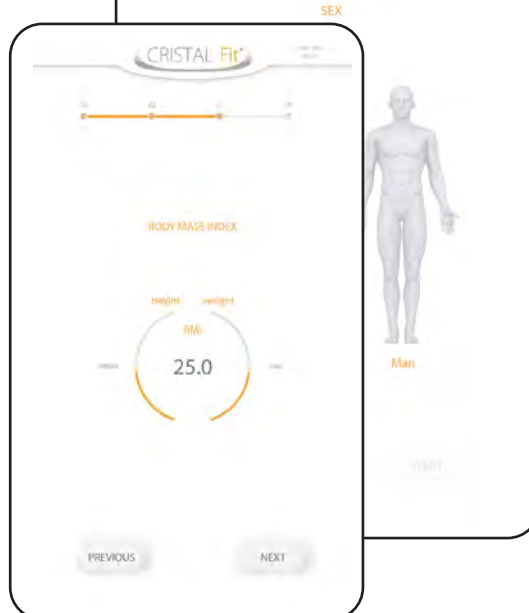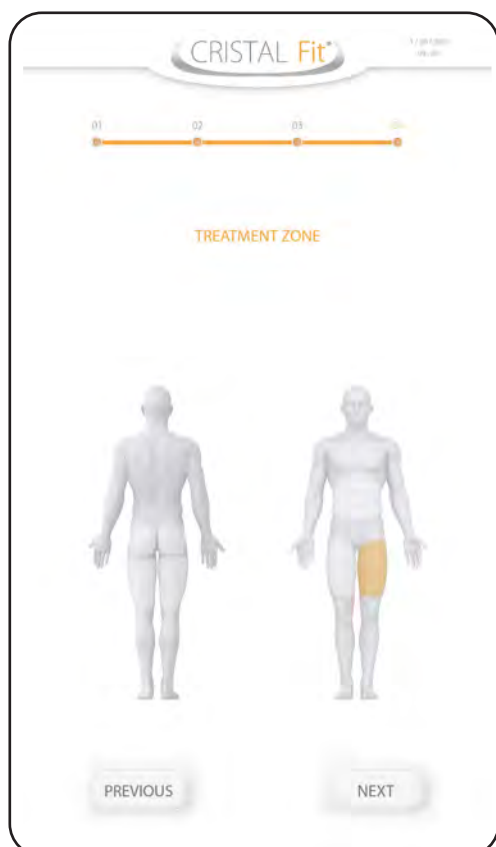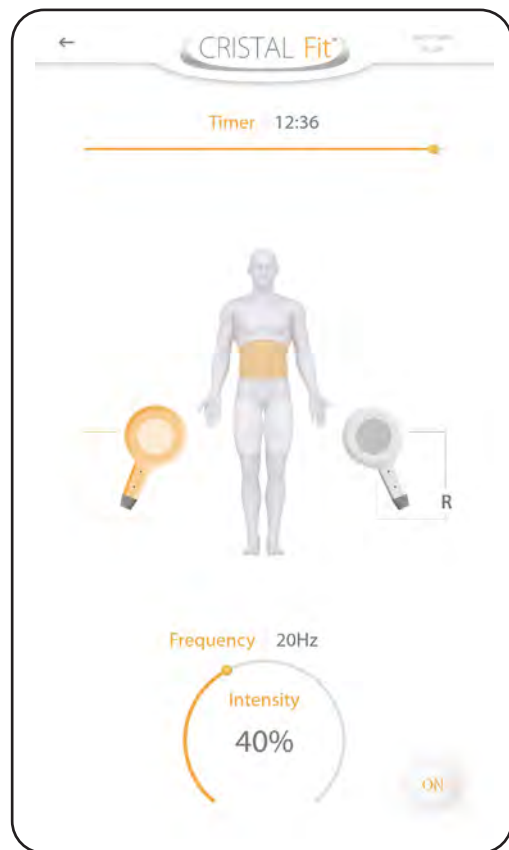

SOFTWARE INTELLIGENCE

Its large screen and intuitive interface make all your **CRISTAL Fit** treatments easier. This device is also equipped with advanced features to support you throughout the treatment plan:

- **Patient records** saved directly on the interface;
- **Prerecorded protocols** according to the area to be treated;
- Ultra-precise **parameters** adjustment (treatment duration, frequency, intensity) for optimal results.






Impressive RESULTS

In only **6 to 8 sessions**, we usually notice a **significant increase in muscle mass** in patients treated one to two months after HI-EMMS treatment.

@DR.AZOULAY

Before

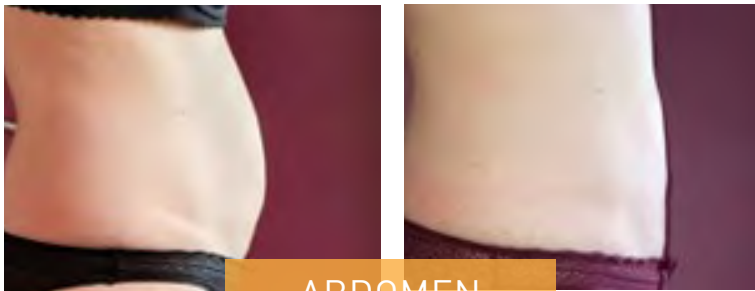

ABDOMEN

After

1 CRISTAL Pro® session
and 4 CRISTAL Fit® sessions

@DR.AZOULAY

Before



ABDOMEN

After

1 CRISTAL Pro® session
and 6 CRISTAL Fit® sessions

@REGENEVA

Before



ABDOMEN

After

1 CRISTAL Pro® session
and 6 CRISTAL Fit® sessions

@REGENEVA

Before



ABDOMEN

After

1 CRISTAL Pro® session
and 6 CRISTAL Fit® sessions



CRISTAL Fit, STRONGER

THANKS TO HI-EMMS TECHNOLOGY





With over 600 installations in the world and over 1 000 000 treatments carried out, CRISTAL is the European standard in Body-Layering®.

The CRISTAL range is growing with the addition of the CRISTAL Fit High Intensity Electromagnetic Muscle Stimulation device that works on the muscle for a visibly more shapely figure.



Deleo S.A.S.
239 via Nova
Pôle d'Excellence Jean Louis
83600 Fréjus
T. 04 94 45 83 75 | contact@deleo.fr
www.deleo-medical.com



BROCHURE DOCTEUR-CRISTAL-FIT-FR-v2.5-290922

FRENCH
TECHNOLOGY

riyadh	+966 11 465 0371	jeddah	+966 12 639 8200	madina	+966 14 813 2302	khobar	+966 13 894 5015
qassim	+966 16 326 5404	khamis mushait	+966 17 237 0072	dubai	+971 4 451 4455	abu dhabi	+971 2 674 6493
kuwait	+965 2 225 0206	manama	+973 1733 1046	doha	+974 4441 6893	muscat	+968 24 297 816
beirut	+961 1 998 189	amman	+962 6 553 9977	cairo	+20 2 23067378	other countries	+971 4 451 4455

imdad : امداد | Qualified Solution